

Ceramspray

Application:

Separating spray for ceramics. To insulate models from applied ceramic mass.

Mode of application:

Shake well before use. Wait for app. 1 min after spraying from a distance of 5-8 cm, the spray a second time. Apply only short bursts, avoid puddles, e.g. inside alveolae. Application by brush is also possible.

Quality:

Diaswiss Ceramspray safely insulates model surfaces from applied ceramics and allows easy lifting of wax modellings for the ensuing firing. No risk of cross-contamination as with brush bottles. Liquid burns without residue.

Risiks:

Avoid inhaling spray vapor.

Storage:

- Container is under pressure.
- Do not expose to direct sunlight.
- Protect from heat in excess of 50°C.

Disposal:

Together with general lab waste (metal can).

Manufacturer:

DIASWISS S.A. * Rte de St Cergue 293 * CH-1260 Nyon * Tel: +41 22 301 56 00

Rev. 11/14