

Diabrush ceram Polishers

Application:

Polishing of natural teeth and dental filling materials. To be used only on teeth and filling/restorative materials, danger of injury if used on soft tissue.

Mode of application:

For reprocessing (cleaning, disinfection and sterilization) see the separate instructions for reprocessing.

For best results please observe the recommended speeds of 1.000 to 5.000 rpm at all times. Exceeding recommendation speed leads to increased risk of injury. Excessive heat may cause damage to the dental pulp. Only use within the limits of the instructions for use. Please observe the instructions for use and recommendations of the handpiece manufacturer. We recommend wearing protective gear such as mouth protection as well as safety goggles. Polishing time max 15 sec., intermittent pressure and application. Water cooling if desired. Additional polishing medium is not necessary.

Quality:

Shanks of nickel-plated brass, fibres charged with abrasive particles.

Risks:

Provided the instrument is used correctly the shank does not come in contact with the mucosa, therefore no allergic reactions have been reported.

Disposal:

Dispose of used instruments together with general clinical waste.

Manufacturer:

DIASWISS S.A. * Rte de St Cergue 293 * CH-1260 Nyon * Tel: +41 22 301 56 00

Rev. 01/17

CE1250